	Shandong Intco Medical Products Co., Ltd.	
	Instructions for use	File No: CE-02-02-02
		Version: A/0
		Page: 1 of 2

Instructions for use

【Product】 Sterile Nitrile Exam Gloves

【Model】 XS、 S、 M、 L、 XL、 XXL

【Manufacturer and address】 Shandong Intco Medical Products Co., Ltd.

Qiwang Road No.9888, Naoshan Industrial Park, Qingzhou, Shandong, China.

【Contact】 TEL: +86-400-050-6868 Fax: 0536 - 6136999

【Product function】 Breaking force: $\geq 6\text{N}$


【Intended use】 It is mainly used for medical examination and plays the role of isolation and protection.

【Instructions for use】 Donning & doffing gloves in a proper way is a skill that needs to be practised by healthcare workers and others that use gloves. Donning must be performed in the correct order to prevent transmission of infections. Keep hands clean before donning gloves. When removing the gloves, avoid allowing the outer surface of the gloves to come in contact with your skin, because the surface may have been contaminated with blood and other body fluids. Avoid snapping, as this may cause contaminants to splash into your eyes or mouth or onto your skin or other people nearby.

【Storage Conditions and methods】 The product should be stored in a cool and dry room with relative humidity $\leq 80\%$ and temperature below 40°C , free of corrosive gases and well ventilated.

【Contraindications, precautions, warnings and suggestions】

- Should read the instructions carefully before using, according to the correct method of use;

	Shandong Intco Medical Products Co., Ltd.	
	Instructions for use	File No: CE-02-02-02
		Version: A/0
		Page: 2 of 2

- This product for ethylene oxide sterilization, only for the use of disposable, ban repeated use; Do not use if the package is damaged.
- People with allergies such as a skin rash or abnormal feeling, should immediately stop using it.
- Avoid such as strong acid, strong alkali and organic solvent of gloves may cause performance degradation or damage to the intensive study drug.
- This product sterilization is valid for three years, please use within valid;
- Products please click medical waste disposal after use.

【Valid】3 year.




Name: Shandong Intco Medical Products Co., Ltd.

Address: Qiwang Road No.9888,Naoshan Industrial Park, Qingzhou City, Shandong Province, China..

Mail: xujuan@intco.com

SRN: CN-MF-000002100

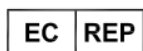

Name: Lotus NL B.V

Address: Koningin Julianaplein 10, 1e Verd, 2595AA, The Hague, Netherlands

Mail: peter@lotusnl.com

Tel: +31644168999

SRN: NL-AR-000000121