	Shandong Intco Medical Products Co., Ltd.	
	Label and Language Information	File No: CE01-02-02
		Version: A/0
		Page: 1 of 2

Instructions for use

【Product】 Sterile Latex Exam Gloves

【Model】 XS、 S、 M、 L、 XL、 XXL

【Manufacturer and address】 Shandong Intco Medical Products Co., Ltd.

Qiwang Road No.9888, Naoshan Industrial Park, Qingzhou, Shandong, China.

【Contact】 TEL: +86-400-050-6868 Fax: 0536 - 6136999


【Product function】 Breaking force: $\geq 6N$

【Intended use】 Medical examination and diagnostic treatment procedures to prevent cross-infection between patients and users.

【Instructions for use】 Donning & doffing gloves in a proper way is a skill that needs to be practised by healthcare workers and others that use gloves. Donning must be performed in the correct order to prevent transmission of infections. Keep hands clean before donning gloves. When removing the gloves, avoid allowing the outer surface of the gloves to come in contact with your skin, because the surface may have been contaminated with blood and other body fluids. Avoid snapping, as this may cause contaminants to splash into your eyes or mouth or onto your skin or other people nearby.

【Transportation and storage】

- Gloves must be covered during transportation and placed in a dry place, free from moisture.
- Gloves should be stored in a well-ventilated, cool and dry warehouse on the shelf 20cm above the ground, should avoid the direct exposure of sunlight or ultraviolet components of strong artificial light, there is no ozone generating device in the

	Shandong Intco Medical Products Co., Ltd.	
	Label and Language Information	File No: CE01-02-02
		Version: A/0
		Page: 2 of 2

warehouse, and should not be in contact with oil, acid, alkali, copper, manganese and other harmful rubber substances.

● The glove is valid for three years from the date of production, and the production date is marked on each small package.

● This product should be stored in a dry, light and heat proof place.

【Contraindications, precautions, warnings and suggestions】

● This product is sterilized, for one-time use only, it is prohibited to reuse; Do not use if the package is damaged.

● Before use, please read the instructions carefully and use in the correct way;

● The sterilization period of this product is three years, please use within the validity period;

● Avoid coming into contact with petroleum-based lubricants, such as Que oil, mineral oil and petroleum jelly, before or during the use of the product. If there is a possibility of contact, please pay attention to the glove damage and replace the glove in time.

● After using this product, please dispose the waste according to the requirements of the hospital or environmental protection department.

● People allergic to natural rubber latex materials are prohibited.

【Valid】 3 year.