

Instruction for Use



1. Product name

Gel mats

2. Specification/Main contents

Model	Size	Model	Size
CGM0616	6"×16"	CGM2031	20"×31"
CGM1012	10"×12"	CGM2263	22"×63"
CGM1216	12"×16"	CGM2435	24"×35"
CGM1218	12"×18"	CGM2735	27"×35"
CGM1224	12"×24"	CGM2745	27"×45"
CGM1620	16"×20"	CGM2835	28"×35"
CGM1818	18"×18"	CGM2865	28"×65"
CGM1843	18"×43"	CGM3535	35"×35"
CGM2024	20"×24"	CGM3555	35"×55"
CGM2028	20"×28"		

Water, glycerol, polyacrylamide.

3. Use Method

1. This product can be used directly .
2. When opening the package, if the product temperature is higher than normal temperature, the cool gel mat can be placed in the refrigerator refrigeration or cool ventilation place, and then used after cooling.
3. In the course of using, if the weight is applied repeatedly at the same position, colloid offset will occasionally occur, which can be smoothed by hand.
4. When the humidity is high, i.e. the room temperature exceeds 35°C, the cooling effect will be reduced. When the cold feeling of the ice mat is insufficient, it can be stored in the refrigerator for 1-3 hours.
5. This product is a non-disposable product and can be used repeatedly.

4. Intended Purpose

Gel mat is used for providing cold therapy for body surfaces.Suitable for local cold therapy in medical institutions or at home. Refer to the warning instructions for restrictions

5. Intended patient population

The gel mat is applied to most patient.But for the elderly,children, We need to pay attention to them.

6. Intended user

It shall only be used by adults who can read and understand the instruction manual.

7. Cleaning method

Wash with warm water and use dishware cleaner if necessary. Then dry the water with a clean towel.

If the device material shows signs of deterioration such as hardening, brittleness, or liquid leakage, do not use it again.

8. Indications

Perform cold therapy on the body.

9. Contraindications

- Raynauds Phenomenon
- Cold Hypersensitivity
- Cardiac Conditions
- High Blood Pressure

10. Undesirable side-effects

Which may cause frostbite for people with sensitive skin to heat. Most time this symptom is transient and typically resolve within a few hours. If the symptom is not alleviate exceed one day, please turn to medical attention.

11. Intended clinical benefits

Reduce body temperature and alleviate discomfort caused by high temperatures.

12.  Warnings

- Check the packaging bag without leakage and product within expiry date before using.
- Individuals with circulatory problems should consult a physician before using this product.

13.  Cautions

- Consult instructions for use.
- Prior to use, store at room temperature.
- Do not puncture outer bag..

14. Disposal

Dispose of contents/container in accordance with local/regional/national/international regulations.

15. Storage conditions

Store at room temperature in a dry place. Avoid direct sunlight.

16. Shelf Life

3 years

17. ImporterName
address**18. Manufacturer**

Jiangsu Intco Medical Products Co., Ltd.

No. 298 Yandunshan Road, Zhenjiang New District 212132 Jiangsu P.R. China

19. EU Authorized RepresentativeMedNet EC-REP C Iib GmbH
Borkstrasse 10, 48163 Muenster, Germany**20. Effective Date:**

2025-09-22

21. Revision:

A/2

22. Contact information

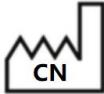
- JIANGSU INTCO MEDICAL PRODUCTS CO., LTD.
- ADD: No. 298 Yandunshan Road, Zhenjiang New District, 212132 Jiangsu P.R. China

Email: sunguihua@intco.com

For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation 2017/745/EU on Medical Devices); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

The contacts of national competent authorities (Vigilance Contact Points) and further information can be found on the following European Commission website: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en.

23.<https://www.intcomedical.com/download.html>Website: www.intcomedical.com**24. Symbol**

No.	Symbol	Meaning
1		Date of Manufacture Country of Manufacture
2		Use-by date
3		Manufacturer
4		EU authorized representative
5		Batch Code
6		CE Marking
7		Caution
8		Do not use if package is damaged
9		Catalogue number
10		Medical device
11		Importer
12		Unique device identifier
13		Model number
14		Keep away from sunlight
15		Keep dry
16		Consult electronic instructions for use