

Instruction for Use



1. Product name

Ice bags

2. Specification/Components

ICB3,ICB4,ICB5,ICB6,ICB7,ICB8,ICB9,ICB10,ICB11,ICB12,ICB13

Cover, Ring, Washer, Polyester fiber bag, Aluminum ring

3. Use Method

Fill the ice bag with cold water of suitable temperature, crushed ice or ice cubes. Securely thread cap onto bag, then apply to areas requiring cold compress.

4. Intended Purpose

Ice bag is used for providing cold therapy for temporarily relief of pain, reduces swelling, and aids in cooling. Suitable for local cold therapy in medical institutions or at home. Refer to the warning instructions for restrictions.

5. Intended patient population

The ice bag is applied to most patient. But for the elderly, children, We need to pay attention to them.

The ice bag shall not be applied to following patients:

Sensory impaired or unconscious patients;

Pregnant women;

Patients situations mentioned in the contraindications;

Do not use on infants.

6. Intended user

It shall only be used by adults who can read and understand the instruction manual.

7. Cleaning method

Wash with warm water and use dishware cleaner if necessary. Then dry the water with a clean towel.

If the device material shows signs of deterioration such as hardening, brittleness, or liquid leakage, do not use it again.

8. Indications

- Relieve local pain and swelling caused by sprains, strains, impacts, bruises, and other injuries, and use within 48-72 hours after the injury occurs;
- Local pain and discomfort caused by toothache and mosquito bites;
- Physical cooling for patients with high fever;
- Local pain and swelling caused by mild burns;
- Used to reduce swelling, pain, and bleeding around the wound after surgery.

9. Contraindications

- Raynauds Phenomenon
- Cold Hypersensitivity
- Cardiac Conditions
- High Blood Pressure
- Superficial Nerves
- Mid-late stage of the wound

10. Undesirable side-effects

Which may cause frostbite for people with sensitive skin to heat. Most time this symptom is transient and typically resolve within a few hours. If the symptom is not alleviate exceed one day, please turn to medical attention.

11. Intended clinical benefits

Relieve pain, control inflammation and swelling, promote hemostasis, physically cool down, and control postoperative swelling.

12.  Warnings

- Check the packaging bag without leakage and product within expiry date before using.
- Do not apply if skin is impaired in any way.
- Don't use on infants.
- Do not freeze in the refrigerator
- Individuals with circulatory problems should consult a physician before using this product.
- Shall pay attention to the elderly, children and people with communication barriers.

13.  Cautions

- Consult instructions for use.
- Fill bag two-thirds full with water. securely thread cap onto bag.
- Do not use for purpose other than intended.
- Do not puncture outer bag.
- Do not use if any damage is found during use.
- Patients should not lie with their full body weight on top of a ice bag.

14. Disposal

Dispose of contents/container in accordance with local/regional/national/international regulations.

15. Storage conditions

Store at room temperature in a dry place. Avoid direct sunlight.

16. Shelf Life

5 years

17. Importer



Name
address

18. Manufacturer



Jiangsu Intco Medical Products Co., Ltd.
No. 298 Yandunshan Road, Zhenjiang New District 212132 Jiangsu P.R. China

19. EU Authorized Representative



MedNet EC-REP C Iib GmbH
Borkstrasse 10, 48163 Muenster, Germany

20. Effective Date:

2025-09-22

21. Revision:

A/2

22. Contact information

- JIANGSU INTCO MEDICAL PRODUCTS CO., LTD.
- ADD: No. 298 Yandunshan Road, Zhenjiang New District, 212132 Jiangsu P.R. China

Email: sunguihua@intco.com

For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation 2017/745/EU on Medical Devices); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

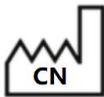
The contacts of national competent authorities (Vigilance Contact Points) and further information can be found on the following European Commission website: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en.

23.



<https://www.intcomedical.com/download.html>
Website: www.intcomedical.com

24. Symbol

No.	Symbol	Meaning
1		Date of Manufacture Country of Manufacture

No.	Symbol	Meaning
2		Use-by date
3		Manufacturer
4		EU authorized representative
5		Batch Code
6		CE Marking
7		Caution
8		Do not use if package is damaged
9		Catalogue number
10		Medical device
11		Importer
12		Unique device identifier
13		Model number
14		Keep away from sunlight
15		Keep dry
16		Consult electronic instructions for use